

**INCREDIBLE**<sup>®</sup>

**ASTRODOME**<sup>®</sup>  
GLASS FACIAL

**OxygenCeuticals Korean  
Glass Astrodome Facial™**

**Oxygen Notes**

AstrodomeFacial

# Oxygen Notes

## LED

- Red → Rejuvenation, aging, brightening, elasticity, 630 nm
- Blue → Acne, pores → 465 nm
- Violet → Post-procedure (relaxing), both lights (Broad wavelength)

## O2

- Healing; Pore size; Vasoconstrictor
- Uses of Ziolite is to attract (N) so the O2 remains (high-concentrated O2)
- Air has 21% O2; You will get 99.9% of that; But with tank is +50%

## Anion

- Kills 99.9% germs; Rainforest waterfall (Lenna effect); Reduces free radicals which is the main cause of aging

## NIR

- 855 nm; stimulates cell activity which helps mitochondria power house of the cell, to increased collagen production, and enhances penetration for both lights

## Dome sessions

- Anion = dry; Mist = wet

## Masks & Finishing

- Pore mask: removes makeup, surface oils, and environmental impurities, ensuring the skin is clean and prepped for treatment.
- HA Gel and Toning Gel: serve as both active skincare and conductive mediums for LED and anion penetration.
- CT mask: (Alginate + Seaweed) Clarifying / acne, Oil control, bacteria reduction
- Serum + cream: balancing the PH of the skin

# Treatment Steps

## 1. Pore Mask

Mix the pore mask (2p) with the cryogenic activator, then stir to make it fluffy.

Cover your eyes, then put on a fluffy mask and start a gentle massage.

Put on face, no brushing, then use a sponge to clean it thoroughly.

## 2. Power Wash (Scaling)

Power-wash the face with a cryogenic activator sprayer. rubbing the skin or a gentle massage on the skin, and at least (2 passes for each area).

Use a sponge on the sides to keep the client from getting wet. (decreasing the skin temperature)

## 3. Change the Activator Bottle to the Oxygen Formula Bottle from the Sprayer

Change the activator to the oxygen formula and start spraying it very closely, 3–5cm from the skin. Use a Q-tip for precision to prevent dripping (no eye and mouth).

## 4. Gel Mask

HA gel + Toning gel - One to one portion, and brush it on the face.

## 5. Dome

Fill the reservoir with 4 cc of the activator for using the dome, and press (LEPA anion or mist + NIR).

Don't forget to cover the eyes with cotton and plastic glasses. (14 minutes of dome time.)

## 6. Clarifying Mask

Clarifying mask + toning gel: Brush it on the face, then cover the eyes and put a fluffy mask (sheet or gauze) over the face.

Then, (alginate mask + distilled water), stir and leave it on the face for 10 minutes.

Then spray the scalp with cryogenic activator. (You can have a normal sheet mask, optional for ease of use, instead of an alginate mask.)

## 7. Finish

Clean face → Put serum → Put cream.

## Important

LED is contraindicated during pregnancy, so turn it off during pregnancy, but it's ok for permanent makeup.