

INCREDIBLE[®]

DEVICES

SkinBrite Mult-IPL[™]

Treatment Parameters

This guide outlines essential Treatment Parameters for safely and effectively operating the INCREDIBLE SkinBrite Mult-IPL, focusing on energy settings, frequency, and the number of passes for optimal results across various skin types.



 HEALTH CANADA LICENCED

 incrediblelaser.com

 647-477-9216

 hello@incrediblelaser.com

Treatment Parameters

Operational Settings	Fitzpatrick	Energy/RF (Js)	Frequency/IPL (Hz)	Number of Passes	
				Stamp	In-Motion
Hair Removal	I	2 - 6Js	3Hz - 5Hz	1 (10 – 20%)	1 Pass
	II	2 - 5Js	3Hz - 5Hz	1 (10 – 20%)	1 Pass
	III	2 - 4Js	3Hz - 4Hz	1 (10 – 20%)	1 Pass
	IV	1 - 3Js	2Hz - 4Hz	1 (10 – 20%)	1 Pass
	V	1 -2Js	2Hz -3Hz	1 (10 – 20%)	1 Pass
	VI	1Js	2Hz	1 (10 – 20%)	1 Pass
Vascular Therapy	I	2 - 6Js	3Hz - 5Hz	1 (10 – 20%)	1 Pass
	II	2 - 5Js	3Hz - 5Hz	1 (10 – 20%)	1 Pass
	III	2 - 4Js	3Hz - 4Hz	1 (10 – 20%)	1 Pass
	IV	1 - 3Js	3Hz - 4Hz	1 (10 – 20%)	1 Pass
	V	1 -2Js	2Hz -3Hz	1 (10 – 20%)	1 Pass
	VI	—	—	—	—
Skin Rejuvenation	I	2 - 6Js	3Hz - 5Hz	1 (10 – 20%)	1 Pass
	II	2 - 5Js	3Hz - 5Hz	1 (10 – 20%)	1 Pass
	III	2 - 4Js	3Hz - 4Hz	1 (10 – 20%)	1 Pass
	IV	1 - 3Js	2Hz - 4Hz	1 (10 – 20%)	1 Pass
	V	1 -2Js	2Hz -3Hz	1 (10 – 20%)	1 Pass

	VI	1Js	3Hz - 5Hz	1 (10 – 20%)	1 Pass
Pigmentation Therapy	I	2 - 6Js	4Hz - 5Hz	1 (10 – 20%)	1 Pass
	II	2 - 5Js	4Hz - 5Hz	1 (10 – 20%)	1 Pass
	III	2 - 4Js	3Hz - 4Hz	1 (10 – 20%)	1 Pass
	IV	1 - 3Js	3Hz - 4Hz	1 (10 – 20%)	1 Pass
	V	1 -2Js	2Hz - 3Hz	1 (10 – 20%)	1 Pass
	VI				
Acne	I	2 - 6Js	4Hz – 5Hz	1 (10 – 20%)	1 Pass
	II	2 - 5Js	4Hz – 5Hz	1 (10 – 20%)	1 Pass
	III	2 - 4Js	4Hz – 5Hz	1 (10 – 20%)	1 Pass
	IV	1 - 3Js	3Hz – 4Hz	1 (10 – 20%)	1 Pass
	V	1 -2Js	2Hz – 3 Hz	1 (10 – 20%)	1 Pass
	VI	1Js	2Hz	1 (10 – 20%)	1 Pass

Treatment Application Method

For **Hair Removal**, use the **SHR In-Motion** method for Body, and the **Stamp Method** for Face & Brazilian:

- *In Motion Method* - slide the handpiece probe continuously over the skin in a slow, controlled motion. 1 pass ONLY, 10 – 20% overlap.
- *Stamp Method* - 1 pass ONLY, 10 – 20% overlap. Apply pressure with each pulse.
- The first three flashes should be directed toward the ground to ensure consistent energy output before beginning the treatment.

For **Skin Rejuvenation**, apply the **IPL (DPL)** system to target specific skin conditions (e.g., Pigmentation, Fine Lines, Acne).

- Caution when using the In Motion Method with SR. Generally, SR is used for Face, hands and spot treatments.
- Ensure even energy distribution with 10–20% overlap across the areas. DO NOT Stack pulses. Use light pressure on the face.

During Treatment

Contact with the Skin: When applying the crystal to the skin, apply it in motion to prevent it from sticking.

Button Operation: Use the handpiece button or foot pedal to initiate flashes. Ensure consistent energy output by pressing continuously.

INCREDIBLE[™]
DEVICES

INCREDIBLE THERAPEUTIC SYSTEMS

 11990 224 Street, Maple Ridge,
BC V2X 2X5, Canada

 647-477-9216

 incrediblelaser.com

 hello@incrediblelaser.com

 facebook.com/incrediblelaser

 instagram.com/incrediblelaser

 tiktok.com/@incredibledevices

 dailymotion.com/incredibledevices

 **HEALTH CANADA LICENCED Medical Devices**

All products are compliant with Health Canada Safety regulations.